

DuoTrain - Shoulder immobilizer

ART.NO 20119

The DuoTrain shoulder immobilization provides comfortable support and relief to ease pain-related shoulder problems. Extra strong Velcro straps, wear plates, allow DuoTrain fastenings to be opened and closed repeatedly, which is why this sling is highly recommended for use in training. The shoulder strap has Velcro fastenings and is adjustable as regards both height and length (along the sling support). Adjustable immobilisation strap around the body. The inside of the sling and shoulder strap are lined with terrycloth. Suits both the left and the right arm.

Indications: For relieve/immobilization/fixation of shoulder/arm. In case of trauma, emergency or postoperative care.

Contraindications: For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

Material: Polyester 80%, Polyamide 13%, Polyurethane 7%. Latex free.

Washing instructions: Close the Velcro fastening before washing. Machine wash at 40°C; can be washed at 60°C if required. Ideally, use a washing net.

Measurement: Elbow to finger tips in cm.

Size	Length forearm in cm	Art.no
Small	35-41	20119-06-40
Medium	40-44	20119-06-50
Large	43-49	20119-06-60
X Large	48-55	20119-06-70

Size:

Small – for children/very small adults.

Medium – for small women.

Large – for men and women

X-large – for tall/large adults.



1. Secure the wear plates in the desired position.

2. Use the D-ring to adjust the length of the immobilisation strap.



MD Class I CE