

# Saga - Prosthesis-bra

ART.NO 31122

Wireless comfortable prosthesis-bra in soft micro fiber material. Supportive cups with pocket for padding/breast form. Wide and upholstered adjustable shoulder straps. High cut neckline and high support under the arm gives a nice fit and support. Hook-and-eye closure in back 3x3 rows.

**Size:** Saga is available in sizes: 75 B-E, 80-105 A-E.

**Indications:** To be used after various types of breast surgery. For breast surgery with a need for a prosthesis. Prosthetic pocket in both cups.

**Contraindications:** None known.

**Materials:** Polyamide 85%, Elastane.15%. Latex free.

**Washing instructions:** Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). Washing in lower temperatures will increase the longevity of the bra. Flat or line dry. Do not tumble dry. Close hook and eye before washing. Use laundry bag. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

**Measurement - how to measure your size:**

Read with size schedule below.

1. Tight circumference measurement around bust = **X**. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80).
2. Loose measurement over the fullest part of the remaining breast, from the centre of the spine to the centre of the sternum = **Y**. Multiply this measurement by two = **Y2**.
3. The difference between **Y2** and **X** decides the cup size (**Y2 - X = cup size**).



Hook-and-eye closure in the back. Pocket for padding/breast form. Wide and adjustable shoulder straps. High support under the arm.



**Size schedule:**

Circumference under bust= X								
Circum. X cm	68-72	73-77	78-82	83-87	88-92	93-97	98-102	103-107
= Size	70	75	80	85	90	95	100	105
Y2-X = Cup size								
Difference (cm)	13	15	17	19	21			
Cup size	A	B	C	D	E			