## Victoria - Post surgical bra

## ART.NO 31146

Post surgical bra that gives optimum support. To be used after breast surgery. The bra can be opened both in front and shoulder straps, allowing easy application and adjustment after surgery and/or during recovery. Rigid cups, adjustable shoulder straps, adjustable front hook-and-eye closure. The wide side panels and racerback design provide good fit and support.

Model	Art.no		
Victoria, white	31146-09		
Victoria, black	31146-00		

Size: Victoria is available in sizes:

White: 65 E-G, 70 D-G, 75 B-F, 80 A-E, 85 A-D, 90 A-B.

Black: 65 D-G, 70 A-F, 75 A-F, 80 A-E, 85 A-D, 90 A-C, 95 A-C.

**Indications:** Post-surgical bra with a measured compression level equivalent to compression class 1. Intended for use after different types of breast surgery.

Contraindications: Infection. Arterial circulatory disorders. High risk of thrombosis.

Materials: Polyamide 70%, Elastane 15%, Cotton 15%. Latex free.

Washing instructions: Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). Washing in lower temperatures will increase the longevity of the bra. Flat or line dry. Do not tumble dry. Close hook and eye before washing. Use laundry bag. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

## Measurement - how to measure your size:

Read with size schedule below.

- 1. Tight circumference measurement around bust = X. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80)
- 2. Loose measurement over the fullest part of the remaining breast, from the centre of the spine to the centre of the sternum = Y. Multiply this measurement by two = Y2.
- 3. The difference between Y2 and X decides the cup size (Y2 - X = cup size).







Racer back. Adjustable front and shoulder straps.



31146-00









## Size schedule:

Circumference under bust= X											
Circum. X cm	63-67	7 68	-72	73–77	78-82	2 83	8-87	88-92	93-97		
= Size	65	7	70	75	80 85		35	90	95		
Y2-X = Cup size									Y		
Difference (cm)	13	15	17	19	21	23	25	27	29		
Cup size	Α	В	С	D	Е	F	G	Н	1	\ \ \x \\ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \	