

# Evelina - Wire bra

ART.NO 31152

Wire bra in soft micro fiber material. Supportive cups, wide and adjustable shoulder straps. Hook-and-eye closure in back 3x3 rows.

**Size:** Evelina is available in sizes: 65 D-G, 70 D-I, 75 D-I, 80 D-I, 85 D-G.

**Indications:** Postoperative Bra.  
**Contraindications:** None known.

**Materials:** Polyamide 90%, Elastane 10%. Latex free.

**Washing instructions:** Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). Washing in lower temperatures will increase the longevity of the bra. Flat or line dry. Do not tumble dry. Close hook and eye before washing. Use laundry bag. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

**Measurement - how to measure your size:**

Read with size schedule below.

1. Tight circumference measurement around bust = **X**. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80)
2. Loose measurement over the fullest part of the remaining breast, from the centre of the spine to the centre of the sternum = **Y**. Multiply this measurement by two = **Y2**.
3. The difference between **Y2** and **X** decides the cup size (**Y2 - X = cup size**).



Hook-and-eye closure in the back. Wide and adjustable shoulder straps. High support under the arm.



**Size schedule:**

Circumference under bust= X						
Circum. X cm	63-67	68-72	73-77	78-82	83-87	
= Size	65	70	75	80	85	
Y2-X = Cup size						
Difference (cm)	19	21	23	25	27	29
Cup size	D	E	F	G	H	I

