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**Tissu 3D, Nylon, Polyester, Mousse PU, Cotton**

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### English Wrist & Thumb Brace

#### Indications

- Thumb ligament injuries (sprains)
- Acute or chronic inflammation of the wrist or metacarpal bones (rhizarthosis)
- Tendonitis and tenosynovitis in the thumb and wrist
- Post-operative immobilization.
- Secondary care after fracture treatment at the thumb or base of the thumb

#### Contraindications

- Severe sprains
- Immediate treatment following fractures

#### Precautions

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The brace should fit snugly and be comfortable without interfering with circulation.
- Ensure appropriate wound care procedures are undertaken prior to application over open wounds or infected areas.

#### Application

- Measure wrist circumference and take the appropriate size.
- Remove the metal stay from the splint and adjust shape according to the patient's anatomy of the wrist. Re-insert it again in the pocket.
- Choose for either short or long plastic thumb stay.
- Insert the straps in the D-rings, pull the brace over the hand and place the thumb in the thumb part
- Tighten the top and bottom straps and attach to the hook. Wrap the center strap around the wrist with moderate tension
- Finally, wrap the strap around the thumb.
- A good positioned splint allows normal finger movement and is not too tight.
- Cut off the excess strap length.

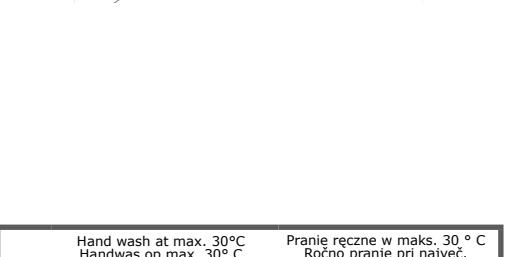
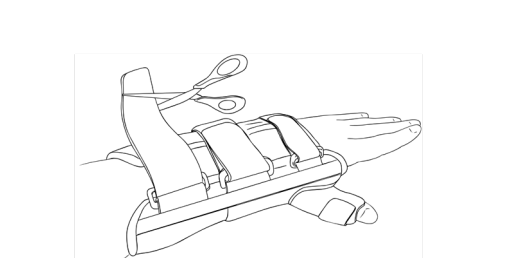
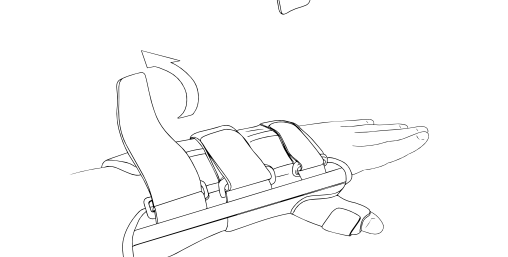
#### Care & Cleaning

Remove aluminium stays from the pocket and fasten straps. Machine-wash at 30°C (delicate fabrics) or hand-wash in lukewarm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure splint is completely dry before re-applying it.

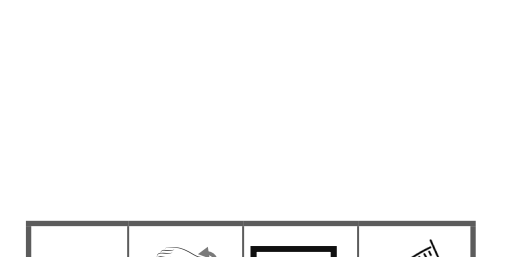
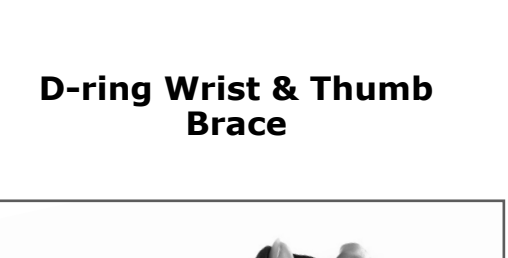
#### ⚠ Important information

- This brace should only be applied in accordance with the instructions mentioned in this manual.
- The product is intended for single patient use.
- No liability is accepted for incorrect product use.
- Guarantee is bound to statutory regulations.

**Spacer Fabric, Nylon, Polyester, PU foam, Cotton**

	Hand wash at max. 30°C Handwas op max. 30° C Lavage à la main jusqu'à 30° C Handwäsche bis 30° C Lavado en mano max. 30°C Lavagem à mão no máximo a 30 °C Lavare a mano max 30° Πλύνετε στο χέρι σε μέγιστη θερμοκρασία 30°C	Pranje rączne w maks. 30 ° C Róčno pranje gri največ 30°C Håndvask ved maks. 30 ° C Käsinpesu max. 30°C Håndvask ved maks. 30°C Róčno pranje gri največ 30°C A se spála manuál la maxim 30°C	
	Do not bleach Geen bleekmiddel Ne pas blanchir Nicht bleichen No usar blanqueador Não usar lixívia Non candeggiare Μην χηρτορροισίετε χλωρίνη	No wybielać Ne belite Ei saa valkaista Må ikke blekes Ne belite A nu se utilizez înălbitor لا تستخدم البلیف	
	No chemical cleaning Niet chemisch reinigen Pas de nettoyage chimique Keine chemische Reinigung No lavar en seco sem limpeza qu Non utilizzare detergenti chimici χηρρίσ χημικό καθαρισμό	Nie czyszcć chemicznie Brez kemičnega čiščenja Ingen kemisk rengaring Ei kemiallista puhdistusta Ingen kjemisk rengjaring Brez kemičnega čiščenja Fără curătare chimică لا يوجد تنظيف كيميائي	
	Do not tumble-dry Niet in droogtrommel droegen Ne pas sécher en sèche-linge Nicht im Trockner trocknen No usar secadora Não utilizar máquina de secar Non asciugare nell'asciugatrice Μην στεγνώσετε σε στεγνωτήριο	Nie suszyć w suszazce Ne sušite v sušilnem stroju Ikke tørretumbles Ei rumpkuuvaista Må ikke tørkes i tørketrommel Ne sušite v sušilnem stroju A nu se usca în uscător لا تجف في الميف	
	Do not iron Niet strijken Ne pas repasser Nicht bügeln No planchar Não passar a ferro Non strare Μη σιδερώετε	Nie prasować Ne likati Må ikke stryges Ei saa silttää Må ikke strykes Ne likati A nu se călca لا قم ابركي	

		
	12-14 cm 11-0047-XSL	X-Small
	14-16 cm 11-0047-SL	Small
	16-18 cm 11-0047-ML	Medium
	18-20 cm 11-0047-LL	Large
	20-23 cm 11-0047-XLL	X-Large
	12-14 cm 11-0047-XSR	X-Small
	14-16 cm 11-0047-SR	Small
	16-18 cm 11-0047-MR	Medium
	18-20 cm 11-0047-LR	Large
	20-23 cm 11-0047-XLR	X-Large

<b>UK CA</b>	<b>CH REP</b>
Poise Medical R.A. Consultants Marshall House, Suite 21/22/23/24/25 124 Middleton Road SM4 6RW Morden, Surrey +44 (0)79 3590 6108 info@poisemedicalra.co.uk	MP MEDICAL PARTNER AG Am Rosenberg 2 CH-9410 Heiden +41 (0)71 543 19 30 www.medicalpartner.ch

<b>Orthobroker</b> bv Ter Stratenweg 11 BE-2520 Ranst +32 33851075 www.braceid.com info@orthobroker.com	 
	