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Indicações

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English Wrist Brace

Indications

- To provide support for inflammatory conditions and soft tissue injuries, at the wrist (tendonitis, sprains and strains).
- Prevent further trauma caused by repetitive activities (overuse injuries).
- Secondary care after fracture treatment.
- Rheumatic conditions with chronic instability.

Contraindications

- Severe sprains.
- Immediate treatment following fractures.

Precautions

- If unusual swelling, skin discoloration or discomfort occurs, stop using and consult a healthcare professional.
- The brace should fit snugly and be comfortable without interfering with circulation.
- Ensure appropriate wound care procedures are undertaken prior to application over open wounds or infected areas.

Application

- Measure wrist circumference and take the appropriate size according the reference table.
- When required, remove palmar aluminum stay from the brace and adjust shape according to anatomy of the wrist, or specific resting position requirements. Reinsert it into the pocket.
- Open all straps and pull the brace over the hand with the thumb through the designated opening.
- Close the center strap around the wrist with moderate tension, followed by top and bottom straps.
- A well-positioned brace allows normal finger movement and is not applied too tightly.

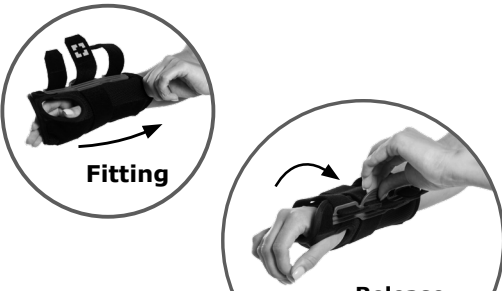
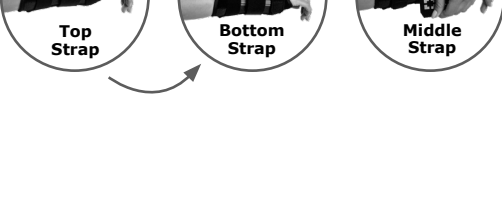
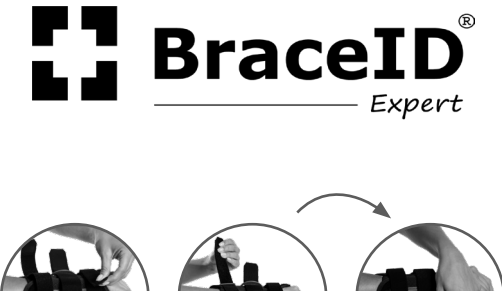
Care & Cleaning

Remove aluminum stay from the pocket and fasten straps. Machine wash at 30°C (delicate fabrics) or hand wash in lukewarm water and mild detergent. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the brace is completely dry before re-applying it.

△ Important information

- This brace should only be applied in accordance with the instructions mentioned in this manual.
- The product is intended for single patient use.
- No liability is accepted for incorrect product use.
- Guarantee is based to statutory regulations.

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