

Multi - Back support

ART.NO 50240

Multi is a shapeable corset that provides comfortable support and secure compression. It can be adapted to suit various needs and body shapes, for both men and women. Suitable for pain conditions in the lumbar spine, hip and pelvis. Also to ease residual pelvic pain after pregnancy. Multi is intended for positioning low down over the hips/pelvis, or higher up around waist height. Multi features broad Velcro straps on the front in order to ensure a snug fit over the hips and around the waist. The elastic straps can be adjusted to provide increased stability and support. The stable/non-elastic material in the front section makes the brace ideal for applying support and stability to the abdominal region. Thin, highly elastic and flexible material in the sides allows Multi to be shaped to suit 'wasp waists' and round stomachs alike. The material used in the back section is both airy and comfortable. Plastic springs in the back section help provide stability. The corset is reversible, so it can be fastened on both left and right sides. Color black.

Indications: Lumbar/lumbosacral back pain.

Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

Material: Polyester 42%, Polyamide 40%, Elastic 13%, Polyurethane 5%. Latex free.

Washing instructions: Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

Measurement: Hip measurement in cm. If the brace is to be used at waist height, use the waist measurement rather than the hip measurement.

Size	Measurement Hip cm	Art.no	Height center front	Height center back
Small	80-95	50240-00-40	15-18 cm	20 cm
Medium	90-105	50240-00-50		
Large	105-115	50240-00-60		
X Large	115-125	50240-00-70		
2X Large	125-135	50240-00-80		



MD Class I CE