

BirkEasy - 3-point brace

ART.NO 50351

Hyperextensions orthosis that are used to limit/control flexion and lateral mobility in the thoracolumbar part of the spine.

Easy to adjust using a screwdriver:

- The lower bracket can be positioned in a moving or a fixed mode. When in moving position, the orthosis becomes more comfortable, especially when sitting down.
- The upper bracket can be adjusted in its width. It can also be adjusted in different angles to achieve the best fit over the chest at different degrees of lordosis.
- The product is adjustable in height. The padding in the side is split and can be placed in the desired position.

When the orthosis is well fitted, taking it on and off is carried out by using the buckle. For safety reasons, the buckle is spring loaded in its locked position. BirkEasy can be showered as the cushion do not absorb water.

Medical personnel must regularly check that all settings and components/screws are secure during the product's use.

Indications: Fracture, vertebral compression, osteoporosis, dislocation, osteoarthritis.


Contraindications: None known.

Material: Anodized aluminum. PE foam padding, Grey synthetic leather. Polyester straps. Latex free.

Washing instructions: Wipe clean with a soft damp cloth.

Measurement: Hip circumference in cm.

Size	Hip measurement cm	Height	Art. no
X Small	74-86	34-42	50351-09-30
Small	84-98	34-42	50351-09-40
Medium	96-110	40-54	50351-09-50
Large	106-120	40-54	50351-09-60

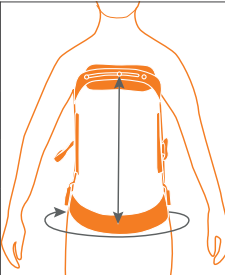


MD Class I CE

