ImmoKid - Knee immobilizer

ART.NO 70510

ImmoKid is a pediatric orthosis used for immobization of the knee joint.

Two anatomically shaped aluminum stays in the back provide good comfort and fit. The lateral stay are attached with Velcro and can be moved for individual adjustment, the stays should be centered medially/laterally along the direction of the femural bone. The stays can be bent/shaped. The orthosis is fixated with three elastic velcro straps. It is possible to cut the length of the material to reduce the circumference of teh orthosis to fit very small knees/legs.

 $\label{location:continuous} \textbf{Indications:} \ \textbf{For immobilization of the knee joint after}$

operation or trauma.

Contraindications: None known.

Size: 2XS-S

Material: 50% Neoprene, 25% Nylon, 10% EVA Laminate, 5% Delrin, 5% Elastic, 5% Stretch UBL. Aluminum stays. Latex free.

Washing instruction: Hand wash max 30°C. Use washing bag. Dry flat. Secure all hook and loop closures before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

Measurement: Circumference of thigh, measure 10 cm/4 inch above knee, from center/mid-patella.

Size	Length of orthosis cm	Circumference cm (fits up to)	Art.no	10
2XS	15	28	70510-00-20	
XS	23	35,5	70510-00-30	
S	27	35,5	70510-00-40	







