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
Compression Garments by NordiCare

Sandra
art no 31147

POST SURGICAL BRA

Compression class 1 (15-21 mm Hg)

OEKO TEX[®] 100 STANDARD
6234-7001 DTI
www.eko-tex.com

This packaging can be recycled as paper and the bag as plastic.

MD Class I
www.nordicare.se
info@nordicare.se
Phone +46 42 35 22 20
Fax +46 42 35 22 21
SE-263 62 Viken
Sollavägen 1,
NordiCare Ortopedi & Rehab AB

2023-02-23

Available colors: 

Compression class 1 (15-21 mmHg)
CC1
Seamless
4-way stretch

Band	size		Cup size				
	cm	inch	A	B	C	D	E
65	26	26	S (Cup A-C)	M (Cup A-C)	L (Cup A-C)	XL (Cup D-F)	2XL (Cup D-F)
70	28	28	S (Cup D-F)	M (Cup D-F)	L (Cup D-F)	XL (Cup D-F)	2XL (Cup D-F)
75	30	30					
80	32	32					
85	34	34					
90	36	36					
95	38	38					
100	40	40					
105	42	42					
110	44	44					

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EN Bra in knitted seamless material without underwires, adjustable shoulder straps, prostesis pocket. Compression equivalent to compression class 1.

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EN Post-surgical bra with a measured compression level equivalent to compression class 1. Intended for use after different types of breast surgery.

Contraindications: Infection. Arterial circulatory disorders. High risk of thrombosis.

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EN If you experience pinching, chafing, or color changes in the skin, remove the bra and contact the product fitter. If you recently went through breast surgery, fitting, prescribing and recommendations of use, should be done by or under the supervision of a medical professional. Serious incidents shall be reported to NordiCare and the competent authority.

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Washing instructions

EN Machine wash at maximum 60°C (140°F), for longer durability, 40°C (104°F) is recommended. Fasten hook- and- eye closure before washing. Use of a laundry bag is recommended. Possible to tumble dry at low heat. Do not iron. If the garment has any blood stains, soak or wash in cold water. Do not use bleach, chlorine or fabric softener as this could cause the material to lose functionality.

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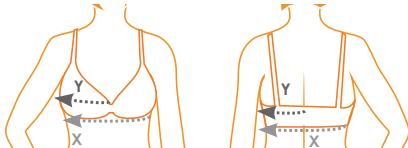
EN Use the size guide and size schedule to decide the right size. The fit should be tight but comfortable. The degree of compression can be adjusted by choosing a larger/smaller size.

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EN 1. Tight circumference measurement around bust = X. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80) 2. Loose measurement over the fullest part of the breast, from the centre of the spine to the centre of the sternum = Y. Multiply this measurement by two = Y2. 3. The difference between Y2 and X decides the cup size. (Y2 - X = cup size)

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Also, see illustration above

Material composition

Polyamide 79%, Elastane 11%, Polyester 10%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.