## Ronja

art no 50531, 50532, 50533, 50536

### SEMI REGID BACK SUPPORT

### LEVEL OF SUPPORT - STRONG

#### Ronja

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**EN Ronja Low/Medium:** Lumbar/lumbosacral back pain.

**Ronja High:** Lumbar/lumbothoracal back pain.

**Contraindications:** Unstable fracture of the back. Sciatica pain of unclear genesis.

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### Heights

Ronja is available in three different heights and in a semi-finished model with edging, elastic stripe and splints included, to finish the product.

Model	Art.no	Height front	Height back
Ronja Low	50531-26	18 cm	28 cm
Ronja Mid	50532-26	20 cm	33 cm
Ronja High	50533-26	23 cm	38 cm
Ronja Kit	50536-06	22 cm	37 cm

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**EN** Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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Ronja kit - 50536

### Measurement

To determine the size, measure hip circumference in centimeters.

### Material composition

Cotton 55%, Polyamide 32%, Elastane 9%, Polyester 4%. Latex free.

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**EN** Machine wash at 40°C (104°F). Remove the splints. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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This packaging can be recycled as paper and the bag as plastic.