50100 LÅG

50100 LÅG

50101 HÖG

Atle

art no 50100, 50101

BACK SUPPORT

LEVEL OF SUPPORT - LIGHT

Atle

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EN Soft back support in breathable, elastic material that provides flexible fit for different body shapes. Elastic straps and moldable stays for adjustable support. Suitable for low back pain, especially suitable for vertebral compression caused by osteoporosis. Available in two heights.

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EN 1. Shape the two white stays to follow the shape of the back and put them into the pocket.
2. Loosen elastic straps. Put the back support in place and fasten the Velcro at mid front over the abdomen, without tightening too hard.
3. Pull the elastic straps firmly outward- forward.

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EN **Atle Low:** Lumbar/lumbosacral back pain.
Atle High: Lumbal/lumbothoracal back pain.
Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

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Heights and Measurement

Atle is available in two different heights in beige and black. To determine the size, measure hip circumference in cm.

Model	Art.no	Height front	Height back
Atle Hög, beige	50101-08	24 cm	34 cm
Atle Hög, black	50101-00	24 cm	34 cm
Atle Låg, beige	50100-08	18 cm	27 cm
Atle Låg, black	50100-00	18 cm	27 cm

Size	Hip Circumference
XS*	65 - 80 cm
S	80 - 95 cm
M	95 - 105 cm
L	105 - 115 cm
XL	115 - 125 cm
2XL	125 - 135 cm

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EN Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Material Composition

Polyester 67%, Polyamide 14%,
Natural Rubber Latex 12%, Elastane 7%.

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EN Machine wash at 40°C (104°F). Remove the splints. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, medical practitioner or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.