
LyftPlus Comfort

art no 50157



ABDOMINAL BINDER

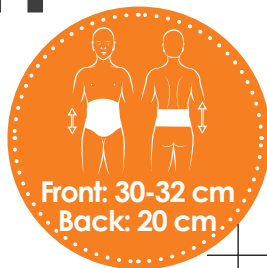2024-06-11

Available color: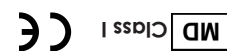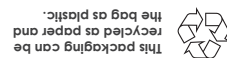

info@nordicare.se,
www.nordicare.se

Phone +46 42 35 22 20,
Fax +46 42 35 22 21

Sweden

Solrosvägen 1,
SE-263 62 Viken

NordiCare[®] +

SE Støtdig abdominal gørdel med formet mageparti for øtt pass en kraftigere buk. Uelastisk front gir effektiv stød øch lyft. Justerbar passform med handtag i sidan. Skenør i ryggen. Rekommenderas direkt etter operasjon eller som permanent gørdel vid brøck øch/eller stomi.

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EN Supportive abdominal binder, shaped front to fit a pendulous abdomen. Non-elastic front provides effective support and lift. Adjustable fit with handle in the side. Recommended directly after surgery or as a permanent binder in case of hernia and/or stoma.

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EN 1. Position the binder and secure the Velcro fastening at the side with the handle. If you have a hernia, it may be preferable to put the binder on while lying down. This will help the hernia to sink in and ensure that the binder provides optimal support.
2. Adjust until comfortable. The binder should give a comfortable support, make sure it is not too tight.

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EN Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN Mobilization after surgery. Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage.

Contraindication: Incarcerated hernia.

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EN LyftPlus Comfort is 30-32 cm high at front and 20 cm high at back. For correct size measure the hip circumference in cm.

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Material Composition

Polyamide 64%, Polyester 30%, Elastane 6%. Latex free.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.