Lyft

art no 50206

ABDOMINAL BINDER

This packaging can be recycled as paper and the bag as plastic.

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EN Elastic abdominal binder with an abdominal shape which provides a comfortable and efficient support/ compression of the abdomen.

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EN 1. Put the binder in place. Close Velcro on the side.
2. Adjust until comfortable. Make sure it is not too tight.

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EN Lyft is an elastic abdominal binder with an abdominal shape to provide support and lift. Provides convenient and efficient support and stability for people with hefty/large abdominal and/or hernia problems. Also used in postoperative care after abdominal surgery, relieves ulcers, prevent incisional hernia and enables early mobilization. **Counter indication:** incarcerated hernia.

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Heights and Measurement

Lyft is 24 cm high at front and 23 cm high at back. For correct size measure the hip circumference in cm.

Size	Hip Circumference
S	80 - 95 cm
M	90 - 105 cm
L	100 - 115 cm
XL	110 - 125 cm
2XL	120 - 135 cm

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EN Prescribing, product fitting and recommendations of use should be done by or under the supervision of a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dryclean. Do not iron. Do not tumble dry.

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