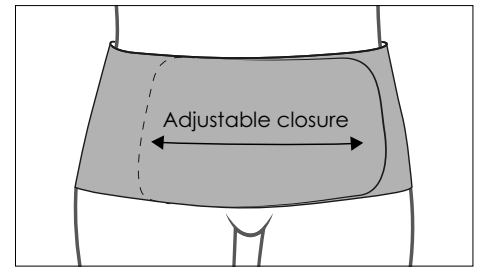
INSTRUCTIONS FOR

Vira Rak™ - BINDER

art no 50230, 50231

Application

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EN 1. Choose the correct size and put the binder in place. Close the Velcro at centre front.

2. Adjust until comfortable. Make sure it is not too tight.

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Material Composition

Polyester 78%, Elastan 12%, Polyamid 10%.
Latex free.

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EN Measure circumference around intended placement. Pick the correct size according to the measurement table.

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Size	Circumference
X Small	70 - 90 cm
Small	80 - 100 cm
Medium	90 - 110 cm
Large	100 - 120 cm
X-Large	110 - 130 cm

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EN For postoperative care after abdominal surgery. To provide relief of surgical wounds and to prevent risk of hernia. For pain relief and early mobilization of patients.

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EN If skin irritation, chafing or changes in skin color occur, discontinue use and contact your doctor or medical practitioner. Prescription and recommended use for the product should be made by a medical practitioner.

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EN Periodically check the condition of the Velcro to ensure they are free from grease and damage. A qualified medical practitioner should supervise use in combination with other medical products.

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EN Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the Velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se.
For further questions, contact your doctor, health practitioner, medical equivalent or
NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.