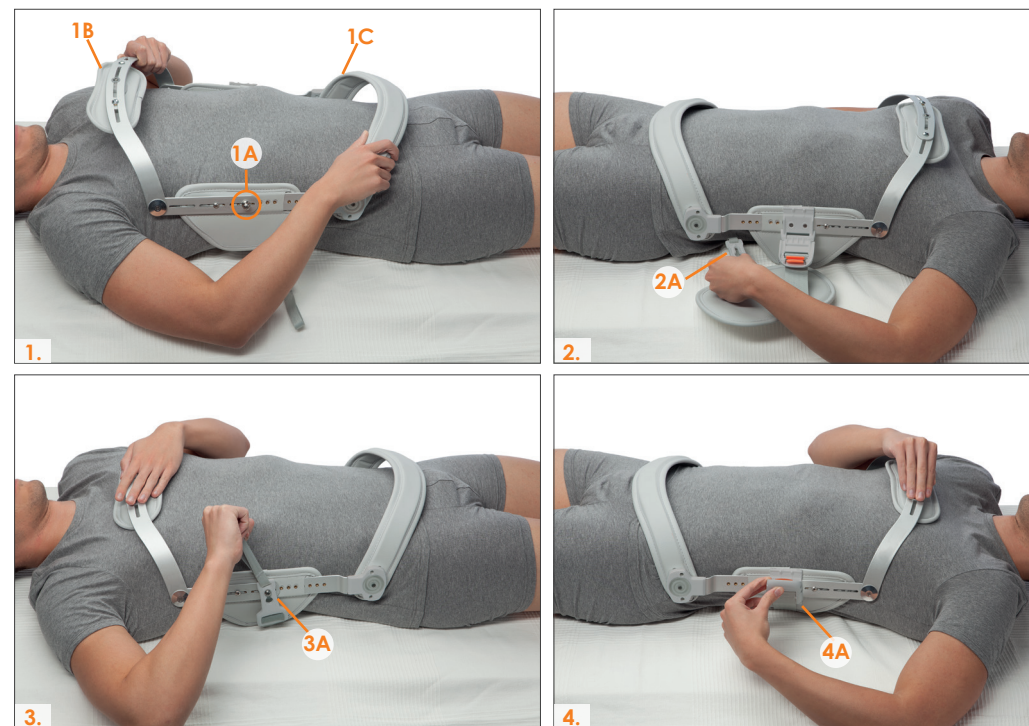
User instruction

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EN When the orthosis has been fitted use the snap lock and the buckle for taking the orthosis on and off (which for safety reasons is spring loaded in its locked position).

1. Open the lever fastening and unhook the strap from the big screw (1A). Place the hyperextension brace on the chest so that the sternal plate (1B) rests on the centre of the chest and pelvic band (1C) rests on the pelvis.
2. Pass the strap (2A) behind the back.
3. Hook the strap to the mushroom head screw (3A).
4. Secure the hyperextension brace using the fastening lever (4A).
5. Regularly check that all screws are tight, contact your tester immediately if you suspect something is not working correctly.

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EN Prescription, device fitting and recommending use of this product may only be done by or under the supervision of a medically trained person. For best results, follow the instructions. Medical personnel must regularly check that all settings and components/screws are secure during the product's use. Consult a healthcare professional when using in combination with other medicines or assistive devices. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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EN Anodized aluminum. Polyester foam padding, Grey synthetic leather (81% Polyvinyl Chloride (PVC), 19% Polyester). Polyester straps. Latex free.

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EN 3-point brace that are used to limit/control flexion and lateral mobility in the thoracic region of the spine. The orthosis is easy to adjust for the best fit.

Fitting and Application instruction

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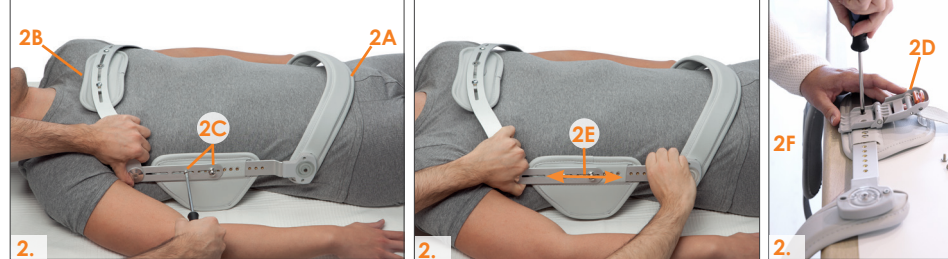
EN The patient should lie down on the back when placing the orthosis on the front of the patient's chest. Check which corrections that are necessary (**1**).

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EN The pelvic bar should be in the middle of the pubis (**2A**) and the sternal plate should be placed in level with the sternum (**2B**). Remove the screws (**2C**) and the buckle (**2D**). Adjust to the right height (**2E**), make sure that both sides have the same length and that the strap is positioned in level of the patients' waist. To make screw adjustments easier, hang the orthosis over an edge (**2F**). When being correct placed, the patient should be able to sit down without any discomfort.

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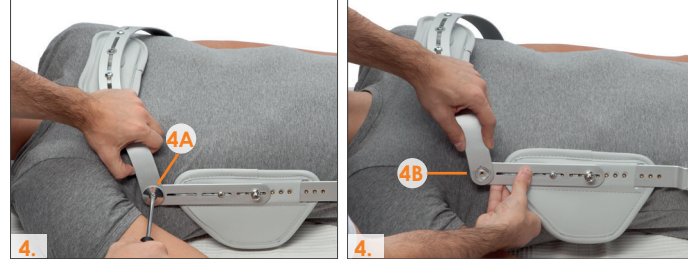
EN Loosen the screws (**3A**), without taking them off. Slide the two overlapping segments to the desired size, the sternum plate should be centered (**3B**). For the extra small size, loose the screws completely and move the outer screws into the track (**3C**) and push the segments together (**3D**).

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EN Remove the screw and remove the metal cover (**4A**) on both sides. Lift up and place it into the desired angle (**4B**), assemble the metal cover and screw. Check that the rail stays vertically centered along the patient's side in the upright position.

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EN The upper bar, pelvic bar and vertical rails can be shaped by bending them over an edge (**5**).

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EN Adjust by removing the middle major screw (**6A**) and place it at the required height and tighten, also place the buckle on the other side to the same level (**6B**).

The lumbar plate can be positioned vertically or horizontally:

Place the strap through the keeper for the intended position (**6C**).

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EN Place the patient on the back, open the snap lever. Pass the strap with the lumbar plate behind the patient's back. Attach the strap to the middle major screw (**7A**). Adjust the length of the strap at the buckle. If the strap is too long, cut it off (**7B**). Fasten the lever into position (**7C**) and make sure that the lumbar plate is centered securely on to the spine. Fasten the clip provided (**7D**).

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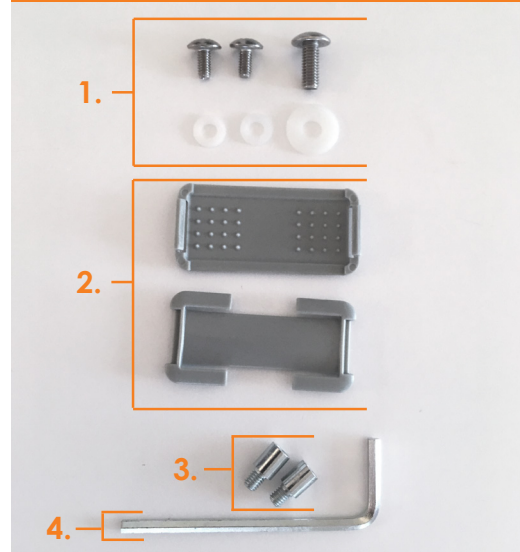
EN The pelvic bar can be secured at an angle of 0 or 18 degrees, alternatively left in an unlocked mode. Place the locking screw (included) in the desired position on each side and tighten (**8**).

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EN
1. Extra screws with associated washers.
2. Strap clip.
3. Locking screws.
4. Allen key.

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