Information

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Fixation band after surgery. For plastic surgery and/or chin/neck liposuction, ear surgery. **Contraindications:** Infection.

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EN Facial Bandage in comfortable material with seamless edges. Adjustable fit and compression. The material is possible to trim/cut to allow optimum comfort and fit.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.

STANDARD
100

1976-298
DTI

This packaging can be recycled as paper and the bag as plastic.

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EN Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). The product will last longer if washed at a lower temperature. Dry flat. Do not tumble dry. Close the Velcro fastening before washing, use a washing net. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

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Materials

Polyamide 76%, Elastane 14%, Polyurethane 10%. Latex free.

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EN Possible to trim and cut the material and straps for optimal fit. For example, trim around the cheeks, chin and neck, or cut holes for the ears.

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