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**EN** 1. Put on the plastic orthosis (1) as a diaper with the child's hip joint in the desired abduction.

2. Fix the plastic orthosis with the textile pant (2) by attaching the pants against corresponding Velcro bands on the plastic orthosis. The long shoulder straps should be positioned against the child's back.

3. Fasten the shoulder straps **A** in the plastic buckle at the belly. Adjust the straps to the desired length.

4. Adjust the four side straps **B** over the belly and fasten with the push buttons.

For the brace to function properly it is important that the correct size is used. Do not push the brace too tight against the child's thighs, the skin is not to be scratched or irritated at the pressure points.

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**EN** Contraindications: There are no known contraindications or risks associated with correct use.

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**EN** Physician or medical professional is to determine the size and how long the brace should be used. The size of the orthosis should be exchanged as the child grows. The ordination of how long the brace should be used per day must be followed. A brace fitted on a child must be checked by its parents on a regular basis. The brace should never be used for other use than the intended. Do not let children play with it. The brace should not be used when the child is being bathed or showered. Ointments or powders should only be used after consultation with physician. Check the velcro straps being free from dirt and that the adhesive has not deteriorated. Regularly check the tape and plastic buckles being intact and functioning.

Prescription, initial fitting and recommendation on how to use this product should be made by medically trained person. If changes in color of skin or other blood circulation related problems and skin irritation occurs, immediately remove the brace. The brace is only for one patient use. Serious incidents shall be reported to NordiCare and the competent authority.

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### Material Composition

Polyethylene 49%, Cotton 29%, Polyamide 13%, Polyester 9%. Latex free.

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**EN** The Pant is machine washable at 40 degrees. Wipe the plastic orthosis with a soft damp cloth, use mild detergent.

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