INSTRUCTIONS FOR USE

AiryROM - POST-OP KNEE BRACE

art no 70521

Instruction for Use

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For more information, see product leaflet or www.nordicare.se. You can also contact your fitter or Nordicare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

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Phone +46 42 35 22 20,
Fax +46 42 35 22 21

info@nordicare.se,
www.nordicare.se

CE This product meets the requirements of Class 1 medical device, non-sterile.

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- EN** 1. Open the knee brace and place it around the leg. If needed the hinges can be re-shaped to follow the contour of the leg.
2. Center the orthosis and hinges along the knee. Close the orthosis, start with the lower closure **2A**, and then the upper **2B**.
3. Make sure the hinges are centered along each side of the knee by adjusting the posterior straps.
4. Close and adjust the Velcro straps. Start with the strap below the knee **4A**, then repeat with the strap just above the knee **4B**, followed by the posterior strap **4C** and finally the two distal straps **4D-4E**.
5. Set the desired range of motion for the hinge. Extension and flexion can be adjusted and locked at 0°, 10°, 20°, 40°, 60° and 90°, or free movement in flexion. Ensure that both hinges have the same setting.

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EN Stabilization of knee joint, post trauma and post operation. Restricting hyperextension.

Material Composition

Polyester 60%, Aluminium 26%, Polyamide 4%, Nylon 4%, Elastane 4%, Polyethylene 2%.

Child	Size	Circumference cm
	X-Small	25-28
	Small	28-34
	Medium	34-40
	Large	40-46
	X-Large	46-56

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EN Circumference around knee, over patella.

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EN User information and care:

Follow the instructions for use. If the product is prescribed by a medical professional, follow instructions as set out in the prescription throughout the rehabilitation period.

Maintenance:

Regularly check that the Velcro straps are free from dirt and that their fastening capabilities have not deteriorated. Avoid using skin cream to prevent a negative impact on the bandage. Do not use the knee brace in the bath or shower.

Always contact your doctor/fitter:

- If you have questions.
- If you are taking medication.
- If you experience side effects such as skin irritation (remove the orthosis).