

# Kim, cutting guide

art.no 51107


## Holes for the ears



1. Mark out both ears with a pen.



2. Cut out the holes.

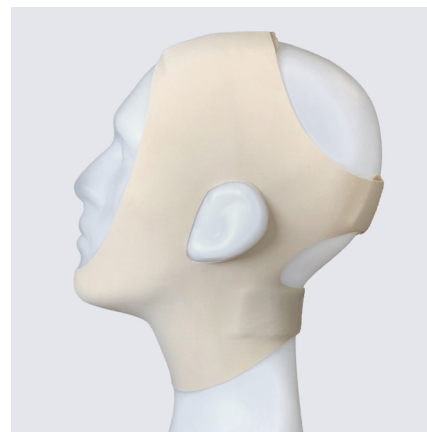


3. Make sure Kim fits well and comfortably.

## Trimmed face line



1. Mark out desired face line.

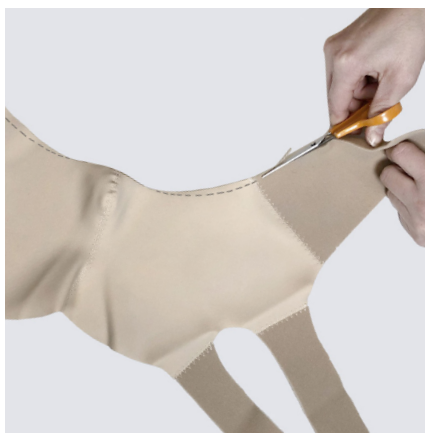


2. Trim along the mark.




3. Make sure Kim fits well and comfortably.

## Removed or trimmed neck

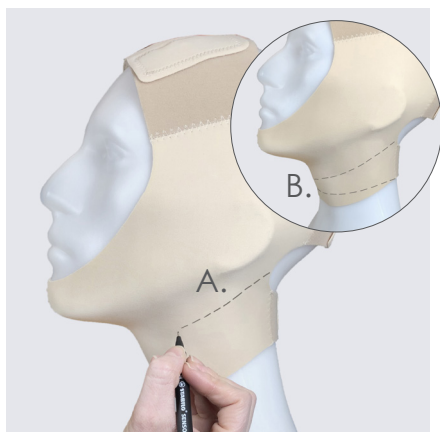

1. Mark out desired neck line. The neck can be removed completely (A) or trimmed to the desired height (B).

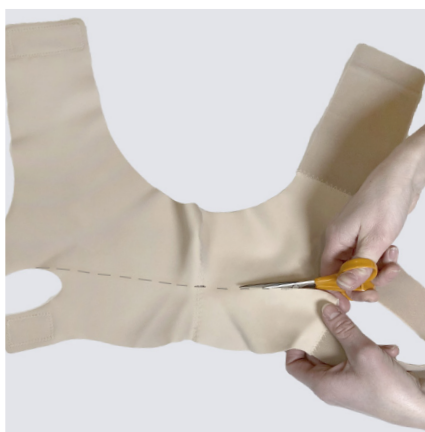

2. Trim along the mark.



3. Make sure Kim fits well and comfortably.

**Please note:** Be careful not to cut off too much material while adjusting the product, rather cut further as you go along. The product should not be worn while cutting.